for Best Care ALCARE

Cellcare 1

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
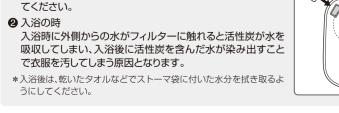
Instruction Manual for digestive stoma

• Keep the instruction manual close at hand and read it again when needed.

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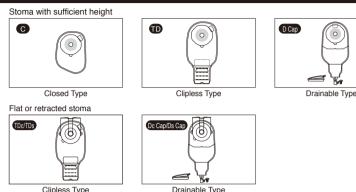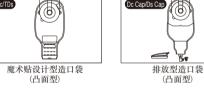
Free-Cut : Start from 1 1 and 2 are not required Pre-Cut Type.

How to Apply

When free-cutting type TDc or Dc caps

Since the shaded area (////) inside range, cut it off before use.

Measure Your Stoma Size Measure your stoma size and ne cutting gauge. Make sure the hole comes center of the gauge. Keep the cutting gauge as your template. * Stoma size may change. Measure once a month to determine your

Turn over the cutting gauge with

film of the faceplate. Trace the hole onto the faceplate. Cut the faceplate 1 to 2mm larger than * Make the inside edge of the faceplate smooth by rubbing with fingers to avoid hurting your stoma.

Make a Hole on Faceplate

Hold the Faceplate Well After application of the face

plate, press with the hands moving outward from the peristoma, to achieve firm contact with the skin.

Excessive air venting

The gas accumulating in the bag through the filter.

aution when using ventilation recoverable filter

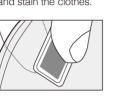
ostomy bag is emitted gradually can be used safely because of its special structure that prevent leakage of faeces in the ostomy

The ventilation recoverable filte

Use the filter cover seal in the package to seal the air vent if you recognize the following conditions. Excessive air venting may result in a vacuum state. In this case, the faeces may not move down into the ostomy bag but remain around the stoma. In order to prevent such a state, you can close the air vent with the filter cover seal in the package. By doing this, gas accumulates in the ostomy

Taking a bath or a shower When bathing, the activated carbon in the filter may absorb water, and it may then leak after bathing

bag. Remove the seal and gently press the bag with your hands to emit the accumulated gas.

Froubleshooting tip •If you notice "difficulty in gas venting", try the following procedures.

Using Cellcare with a trunk belt

* You may find faeces on the filter. Wipe the skin and stays securely in and remove it with a scrubbing motion.

* Pinch the front and back sides of the filter between your thumb and index and D cap, use the Trunk Belt with finger, and compress two or three times.

• For more secure fixing, we Face Plate. For those who use recommend using it with a cap, use the Youcare Trunk Be trunk belt. (you cannot use type TD70 free and D cap free with a belt). Fo details on how to use the belt please refer to the instruction

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Face Plate and the Youcare Trunk ensures that it adheres closely to

Fold up the plate 4 times Fold up the bag end along

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How to open and close <Clipless Type TD/TDs/TDc>

—When It Closes—

-When It Opens-

the Fold Up Line. The Zipper (Hook Side) comes up to the front.

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(Emboss is a marker) by the

●If either Loop or Hook Side of

Zipper gets wet, it may lose adhesion and may soil the

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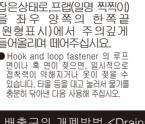
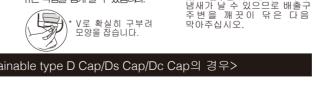
* If any faeces remain around the open-ended area, it may cause the leakage of faeces or smell. Wipe off properly before you

Slowly expose the "

the faeces.

area, to toilet and discharge

How to open and close < Drainable typeD Cap/Ds Cap/Dc Cap>

In case of

Outlet is clogged with remained faeces

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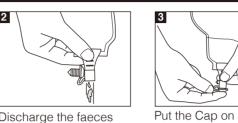
Cellcare 1 can be used as a

open the folded area. Keep shaped plate, open-ended

Hold the outlet upward and carefully remove the Cap so Place Cap on Cap stopper. Aim outlet downward toward toilet that faeces will not splash out. and discharge faeces. Please squeeze the outlet with your tip of fingers to discharge

Hold the Plate upward,

the open-ended area "V

shape by folding Plate. It

will be easier to discharge

shape to form a

and wipe off.

After discharge, wipe out outlet part with tissue or etc. Then put

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and twist the bag around the arm once. * Make sure that the curve of the Clip fits to the body contour.

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leakage from the faceplate.

Avoid long-term stockpiling.

• Do not peel off the release film until use.

area, or in a freezer.

Do not apply ointment or cream around the stoma.

• Do not reapply an ostomy bag once it has been peeled off.

• Warm the adhesive face before application if it is cold.

Do not store in an area of high-temperature, high-humidity

Do not store in direct sunlight area and avoid any pressure

• After removal of the release film, avoid to touch the surface of the faceplate.

 $\bullet \, \mathsf{Do}$ not wash the ostomy bag you are using with water. This may damage the bag and

• Do not fold the ostomy bag that has already been set. The ostomy bag may be perforated

• In the case of free-cutting, cut within the specified valid size range. Cutting beyond the

maximum valid size or using it in a size smaller than the minimum valid size may cause

• Do not keep an excessive amount of faeces in the ostomy bag, and dispose of it appropri-

ately. Excessive storage may cause the faceplate to peel off due to the weight.

Discontinue use if skin becomes irritated. In this case, consult with doctors or nurses.

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E-mail: inquiry@alcare.co.jp Fax: +81-3-5611-7827

Cellcare 1 종류

Peel Off the Release Film After checking that the skin surface is completely dry, peel the release film off the surface of faceplate.

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ALCARE Co., Ltd. 19F, Arcacentral Bldg., 1-2-1 Kinshi, Sumida-ku, Tokyo, 130-0013, Japan